Appendix K

Test Items, Test Manual, and Care Instructions

The biVABA Test Manual is located on the website- <u>www.courses.visabilities.com</u> Recording forms and detailed instructions for each assessment can be downloaded and demonstration videos of the assessments can be viewed at the website.

The Test Battery includes the following items.

Test Charts

LeaNumbers Intermediate Acuity Test Chart LeaNumbers Low Contrast Flip Chart with 10M Optotypes Damato Campimeter and plastic recording form template The Warren Text Card Single Letter Search Simple with scoring template Single Letter Search Crowded with scoring template Word Search with scoring template with scoring template Plain Circles Search Simple with scoring template Plain Circles Search Crowded with scoring template Random Complex Circles Search with plastic target example and scoring template Structured Complex Circles Search with plastic target example and scoring template Design Copy cards (house, flower, clock)

Miscellaneous Items

Penlight	Eye patch occluder
Plastic handheld occluder	Plastic card with 8mm hole
Carrying case	

Not included: red felt tip marking pen, pencil, white 8 x 11 paper, materials to create ScanCourse targets, bookstand, easel

Care Instructions

Hold the test charts by their edges to keep the chart surfaces clean and prevent natural oils on your fingers from degrading the print on the charts. Use the soft tipped pointer to touch the print on the charts. Store the test charts in the carrying case after use to keep the surface of the charts free of dirt. Use warm water and a soft cloth to wipe off the front of the charts. Use only non-abrasive, mild petroleum free dish detergent to clean smudges off of the chart surfaces. **DO NOT** use disinfectant to clean the charts. **DO NOT** scrub the front surface of the charts.

If test items are lost or damaged. Contact craig@visabilities.com for replacement.